




Venipuncture Instructions for T790M-Testing on Liquid Biopsies

- Please follow CLSI H3-A66 guidelines and use **Venofix Safety Kanüle G21** for venipuncture
- Keep the patient's arm in a downward position during blood collection, to prevent a backflow (the tubes contain chemical additives!)
- Please pay attention to the date of expiry on the tube!
- Fill at least two Streck-tubes per patient **completely (9ml)**
- Release the tourniquet once blood starts to flow in the tube
- Remove tube from adapter and immediately mix by gentle inversion 10 times

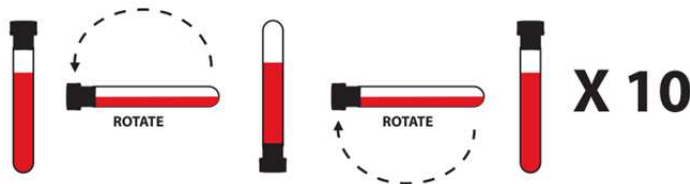

- Do not freeze the tubes
- Streck-tubes can be stored and send at room temperature
- Ship your sample well protected in a mailing container
- We recommend a shipment via a courier service
- Please announce the shipment of your samples in front:
Molecular Pathology +49 30-450 536 035 or via E-Mail:
molpatho-auftraege@charite.de; dido.lenze@charite.de; claudia.vollbrecht@charite.de
- We need the filled request form or a referral letter for medical account:
https://pathologie-ccm.charite.de/diagnostik/molekularpathologie/leistungsspektrum/liquid_biopsies/
- Please also specify the *EGFR* primary mutation on the request form
- To achieve an adequate plasma volume, it may help when the patient drinks sufficiently before blood collection

Recommended Consumables

Article	Description	Company	Order-No.
Cell-Free DNA BCT	Blood Collection Tubes	Streck (Vertrieb in Deutschland über HiSS Diagnostics GmbH)	230256
BloodCulture Adapter Universal	Blood Collection Adapter	Sarstedt AG & Co	14.1209
Venofix Safety (G21 0,8 x 19mm)	Blood Collection Cannula	B. Braun Melsungen AG	4056504-01
Mailing Container with Cap	Protective Vessel for Shipment	Sarstedt AG & Co	78.573

On request, we are pleased to provide you with Streck-tubes for the first analysis.